

FAQS

HOW LONG DOES NEXA LAST?

The Nexa Device is designed to last 8 weeks continuously with battery recharges. It is indicated for multiple patient use so 8 weeks can be divided amongst patients, offering a cost effective NPWT solution.

The Nexa Fluid Container Pack has a volume of 125mls. However it is recommended to replace the pack at least once a week to maintain optimum therapy.

The Nexa Dressing should be changed at least 2-3 times a week, dependent upon the wound state and advice from the clinician.

CAN I USE NEXA OUTSIDE OF THE HOME?

Yes! We have designed NEXA to be used by mobile patients. It is supplied with a carry case and its low, predictable noise won't offer any surprises.

HOW LONG DOES THE BATTERY LAST?

The battery lasts for approximately 10 hours. When the orange low battery light is lit, there is approximately 2 hours remaining. Charging the device takes 4 hours when being used or 2 hours when the device is off.

With first use, the device will require charging for 4 hours.

CAN I SHOWER WITH THE NEXA?

NEXA needs to be disconnected from the dressing prior to showering and left in a dry room. The dressing can stay in place during showering.

WHERE SHOULD I POSITION NEXA?

NEXA must be operated in its upright position but can be used within its carry case or outside of it.

WILL NEXA BE PAINFUL?

Pain thresholds vary between patients the level of pain will depend on the size and type of wound. NEXA starts at a higher speed for 3 minutes to evacuate the dressing gently to reduce the impact of any pain.

ARE THERE ANY ALARMS?

There are no audible alarms on NEXA. This is because the system can manage minor leaks in the dressing and continue to remove exudates. The device has visual indicators to show the battery state and when the Fluid Container is full.